


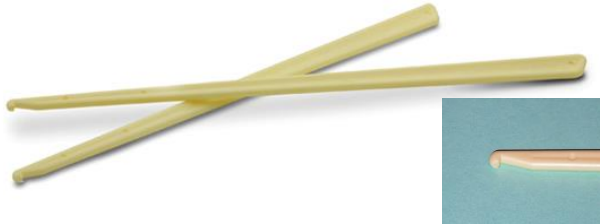


















Obstetrics and Gynecology






Obstetrical and Gynecological Instruments



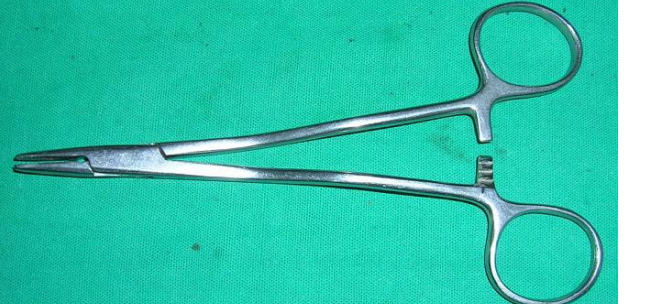

Instruments	Images
Amnihook <ul style="list-style-type: none"> Used to perform artificial rupture of membrane 	
Alli's Forceps <ul style="list-style-type: none"> Use to grasp tough structure like Rectus Sheath Use in LSCS, Hysterectomy 	
Artery Forceps <ul style="list-style-type: none"> Straight end → stay suture Curved end → hemostat 	
Ayre's Spatula <ul style="list-style-type: none"> Use in Pap Smear 	
Babcock's Forceps <ul style="list-style-type: none"> Use to grasp tubular structure like Fallopian tube 	



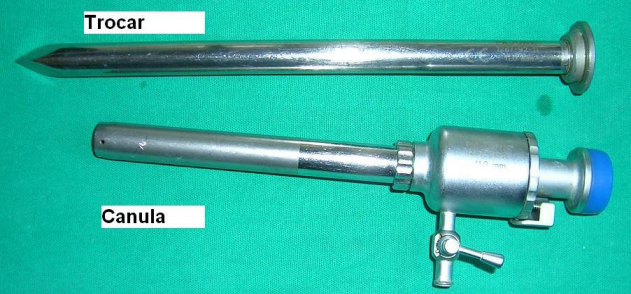

Band Applicator for BTL <ul style="list-style-type: none"> Used for applying silastic bands to fallopian tubes in laparoscopic tubal ligation 	
Bladder Sound <ul style="list-style-type: none"> Use to measure bladder length 	
Cervical Brush <ul style="list-style-type: none"> Use in Pap Smear 	
Cusco's Speculum <ul style="list-style-type: none"> Self-retaining double bladed vaginal speculum. 	

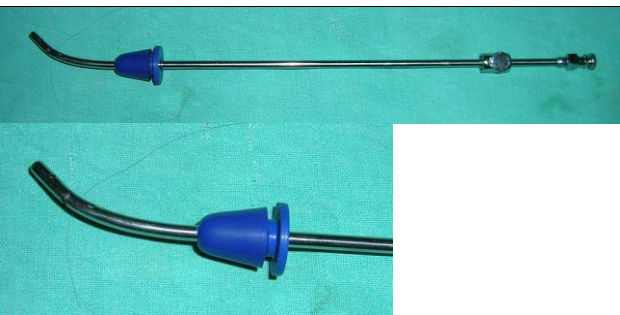



Combined Uterine Manipulator and Cannula for Laparoscopy	
Doyen's Retractor <ul style="list-style-type: none"> Use to retract bladder during LSCS/ Hysterectomy 	
Deaver's Retractor <ul style="list-style-type: none"> Use for retraction of deep structure 	
Episiotomy Scissors <ul style="list-style-type: none"> Used in Episiotomy 	
Foley's Catheter <ul style="list-style-type: none"> Drainage of urine Induction of labor 	
Green Armytage Forceps <ul style="list-style-type: none"> Used as a hemostat in caesarean operation 	

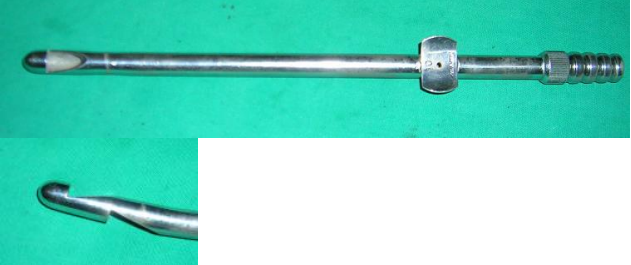


Hegar's Dilator <ul style="list-style-type: none"> Use to dilate the cervix Used in procedure like DnC 	
Kocher's Forceps <ul style="list-style-type: none"> Used to hold Pedicles in Hysterectomy 	
Karman's Syringe <ul style="list-style-type: none"> Used for Menstrual Regulation and endometrial aspiration. 	
Leech Wilkinson Cannula <ul style="list-style-type: none"> Used in tubal patency test like HSG The tip is screwed thru cervix, then dye is injected 	
Needle Holder <ul style="list-style-type: none"> Used to grasp needle during suture 	

Ovum Holding Forceps <ul style="list-style-type: none"> Used to remove POC in miscarriages 	
Purandare's Dilator <ul style="list-style-type: none"> It's a type of cervical dilator 	
Pinard's Fetal Stethoscope <ul style="list-style-type: none"> Used to auscultate fetal heart 	
Pipelle <ul style="list-style-type: none"> Used to take Endometrial sample 	
Rubin's Cannula <ul style="list-style-type: none"> Used for tubal patency test for infertility like HSG 	

Right Angle Retractor	
Sims Anterior Vaginal Wall Retractor <ul style="list-style-type: none"> Used together with Sim's speculum Useful the case of cystocele 	
Sims Speculum <ul style="list-style-type: none"> Used for visualization of vagina and cervix Retract posterior vaginal wall <p>Advantage : wide area for inspection</p> <p>Disadvantage : non-self-retaining</p>	
Sponge Holder <ul style="list-style-type: none"> Used for holding sponge or a gauze piece for painting the area before operation 	

Suction Curette <ul style="list-style-type: none"> Used for first trimester MTP, suction of vesicular mole 	
Shirodkars Circalage Needle <ul style="list-style-type: none"> This is specially designed needle for putting stitch around the cervix. The needle is inserted around the cervix through the opening made in vagina 	
Trocar and Cannula <ul style="list-style-type: none"> Used during Laparoscopy 	
Umbilical Cord Clamp <ul style="list-style-type: none"> Used to clamp the Umbilical Cord 	

Umbilical Cord Scissors <ul style="list-style-type: none"> Cutting the umbilical cord 	
Uterine Curette <ul style="list-style-type: none"> Use for scraping endometrial cavity to obtain sample for histopathology 	
Uterine Sound <ul style="list-style-type: none"> Used to measure Uterine length 	
Tenaculum <ul style="list-style-type: none"> Used for grasping the cervix (Usually anterior lip of the cervix is grasped) 	

Vulsellum <ul style="list-style-type: none"> Used for grasping the cervix (Usually anterior lip of the cervix is grasped) 	
Vantouse	 <div data-bbox="409 581 634 634">Metal Cup</div> <div data-bbox="730 581 955 634">Sialastic Cup</div>
Varies Needle <ul style="list-style-type: none"> used for creating pneumo peritoneum (Putting Air or CO2 in the peritoneal cavity) for laparoscopy 	

Wrigley's Forceps	
--------------------------	---